


What is the Risk?

Electronic devices in both clinical and non-clinical areas can become contaminated with microbes, which can then be transmitted via contaminated hands.

A disinfectant effective against COVID-19 must be used for all Telehealth devices (mobile phones, tablets, and WOWs), and they must be disinfected after every patient use.

Key Principles

- Take the least amount of necessary equipment into a patient room.
- Disinfect all electronics when leaving the room.
- Hand hygiene is the most important factor in infection control and prevention.
 - Perform hand hygiene before accessing the device and after using the device, when working directly with a patient.
 - Unless you are in a patient room requiring gloves (e.g., PUI/COVID+), do not wear gloves when using electronic devices.

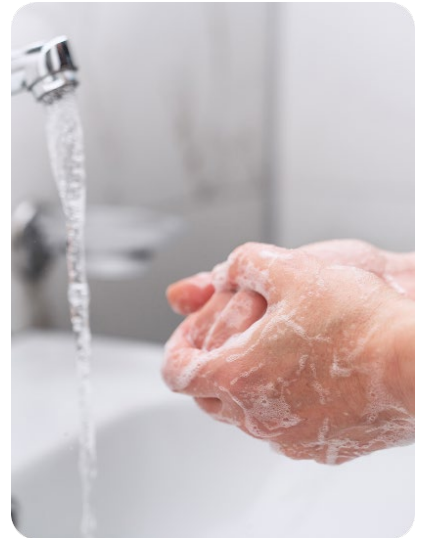

Process

- Perform hand hygiene either by using alcohol-based hand rub or by washing hands with soap and water.
- Disinfect equipment with disinfectant wipes such as Oxivir®, Super Sani Cloth®, Sani Cloth®, Clorox Hydrogen Peroxide®. The type of disinfectant available varies by site. Check the expiration date.
- Follow the manufacturer label and Material Safety Data Sheet (MSDS) sheet regarding use of personal protective equipment.
- Cleaning and disinfection are performed in three steps:
 - 1) Take a disinfectant wipe and squeeze to remove excess fluid. Use a wipe to clean and remove visible soil. Discard wipe.
 - 2) Take a second wipe and squeeze to remove excess fluid. Use the wipe to disinfect following instructions for time surface must remain wet. Discard wipe.
 - 3) Allow surfaces to air dry as per disinfectant instructions.
- Perform hand hygiene either by using alcohol-based hand rub or by washing hands with soap and water.



Appropriate Cleaning Materials

- Follow manufacturer's instructions for use for specific recommendations.
- Use a disposable soft, non-abrasive, lint-free damp cloth or wipe, pre-moistened with a ready to use (RTU) cleaner/disinfectant.
- Never spray products directly onto electronic devices
- An example of disinfectants that may be indicated in the manufacturer's instructions for use:
 - Combination products such as alcohol/quaternary ammonium (e.g., CaviWipes®).

Confirm the disinfectant is compatible with the device and will not cause damage. Prioritize COVID-19 effectiveness over device-disinfectant compatibility.

